

iTero Digital Files

Option 1

- Select “Find a Laboratory” on your iTero scanner
- Connect with Mid-South Dental Lab using our Company ID: 30488
- After connecting to us as a Lab, select “Mid-South Dental Lab” on your scanner when sending files

Option 2

- Call 800-577-8767
- Select Option 1
- Request that Mid-South Dental Lab is added to your scanner. Identify our lab using our phone number 615-790-7637
- After Mid-South Dental Lab has been added, restart your scanner
- After connecting to us as a Lab, select Mid-South Dental Lab on your scanner when sending file